MOPEDIA®

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**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

INDEX

1. CODES	pag.15
2. INTRODUCTION	pag.15
3. INTENDED USE	pag.15
4. EC DECLARATION OF CONFORMITY	pag.15
5. GENERAL WARNING	pag.15
6. GENERAL DESCRIPTION	pag.16
7. ASSEMBLING.....	pag.16
7.1 Assembling RS700-RS702	pag.16
7.1.1 RS700.....	pag.16
7.1.2 RS702.....	pag.17
7.2 Assembling RS705	pag.17
7.2.1 Fixing tank edge	pag.18
8. BEFORE USE	pag.19
9. NOTE FOR USE	pag.19
9.1 Note for use for RS700	pag.19
9.2 Note for use for RS702	pag.19
9.2.1 Independent use	pag.19
9.2.2 Use with assistance	pag.20
9.3 Note for use for RS705	pag.20
10. CLEANING AND DISINFECTION.....	pag.20
10.1 Cleaning	pag.20
10.2 Disinfection	pag.20
11. CONDITIONS OF DISPOSAL	pag.20
11.1 General conditions of disposal	pag.20
12. SPARE PARTS AND ACCESSORIES	pag.20
13. TECHNICAL FEATURES	pag.21
14. WARRANTY	pag.23
14.1 Repair	pag.23
14.2 Spare parts	pag.24
14.3 Exempt clauses	pag.24

I Class Medical Device
According to Directive 93/42/EEC and further modifications

1. CODES

RS700 Swivel shower chair without backrest

RS702 Swivel shower chair with backrest

RS705 Bath board with shower chair

2. INTRODUCTION

Thank you for purchasing a MOPEDIA bath aids by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The bath chair by Moretti is intended for disable patient assistance in bath tub.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the bath aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT use/install the devices in question for any other purpose** than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment, otherwise injury or damage may occur
- Keep the packed bath chair away from heat sources
- SERVICE LIFE- the product use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the bath chair

6. GENERAL DESCRIPTION

RS700-RS702

Adjustable swivel stool with handle with and without backrest. Painted aluminum frame, TPR (thermoplastic rubber) handle and board, TPE central swivel and non-slip seat. Swivel seat with 90 degree position lock at every right angle location. 8 position seat height adjustment by 1" increment.

RS705

Bath board with shower chair. Innovative central bath seat flipped up allow more space for bathing. Realised in PE with aluminum pointed grave, rubber covered clamps and under stripes resist slipping and protect surfaces from scratching.


7. ASSEMBLING

7.1 RS700-RS702

The RS700-702 products are supplied unassembled and consist of:


- Seat (1 pcs)
- Backrest (only for **RS702** model) (1 pcs)
- Handle (1 pcs)
- Legs (4 pcs)
- Screws

7.1.1 RS700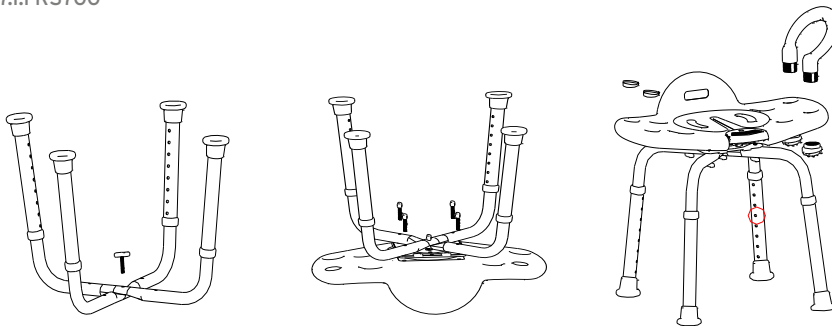

- Put two legs cross to form right angle and screw on the knob in the center hole;
- To screw assembly other knobs with the swivel seat;
- Adjust the seat height;
- Select one end for handle screwed on, and covers up the small lids on the other end;

7.1.2 RS702

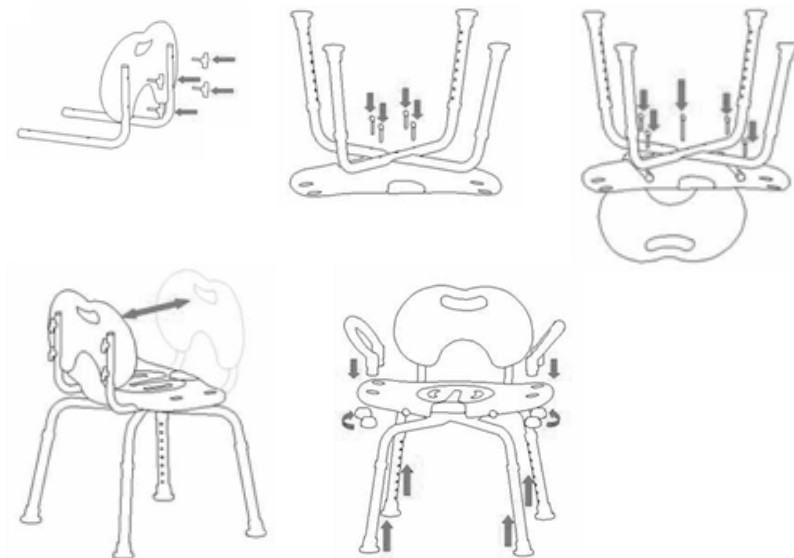7.2 RS705

The bath board with shower chair **RS705** is supplied unassembled and consist of:

- Main Bath Board (1 pcs)
- Grip (1 pcs)
- Knob (2 pz)
- Small Lids (2 pcs)
- 8 position Connecting Rod with Hooks (1 pcs)
- Bath Seat (1 pcs)
- Screws

A. Select one end for handle assembly, and over up the small lids on the other end;

B. Connecting main bath board with bath seat by hooks on each end of rod;

C. Insert four legs into the holes below the bath seat, then assembly completed;

7.2.1 Fitting the Bath Board


A. Place the Bath board on the top of the tub wall with levers "UNLOCK" position;

B. Slide the plates closely as possible to the inside wall of tub;

C. LOCK the levers as shown. Fitting completed.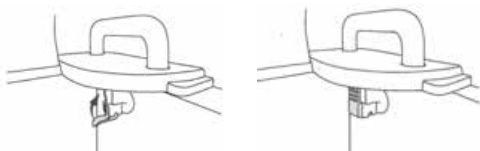


8. BEFORE USE

- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

9. NOTE FOR USE


9.1 Note for use for RS700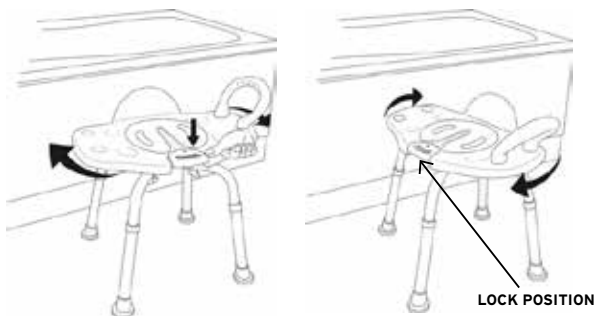

- Adjust the seat height, Sit on, Push down the front bottom to swivel the top seat;
- Turn around right or left with one hand holding on the handle;
- Hand off the front bottom;
- Position lock at right angle location;

9.2 Note for use for RS702

9.2.1 Self Operation

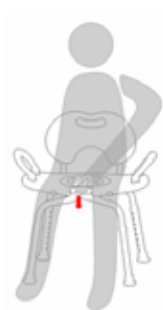- Adjust the seat height, Sit on, Push down the front bottom to swivel the top seat;
- Turn around right or left with one hand holding on the handle;
- Hand off the front bottom;
- Position lock at right angle location;

9.2.2 Nursing Operation

Nursing people take care and operate the seat swiveling from the back bottom for the patient or elderly.

9.3 Note for use for RS705

- A. Sit on the side of the bath board
- B. Move one leg over the bath top wall slowly
- C. Again another leg and move body to the center with one hand on the grip
- D. Stand up to hold the rail and flip up the central seat
- E. Sit down on the chair carefully in the bath tub

10. CLEANING AND DISINFECTION

10.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use.

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If you need to disinfect the device use a common disinfectant cleaner.

NOTE: Never use acids, alkalis or solvents such as acetone.

11. CONDITIONS OF DISPOSAL

11.1 General conditions of disposal




In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

12. SPARE PARTS AND ACCESSORIES





For spare parts only refer to MORETTI master catalogue.

13. TECHNICAL FEATURES

Dimension RS700	 L: 536 mm	 W: 432 mm	 H: 544 mm
 Weight	3,44 kg		
 Max loading weight	136 kg		

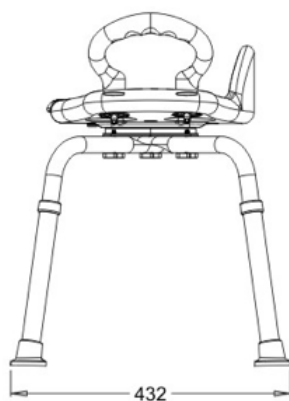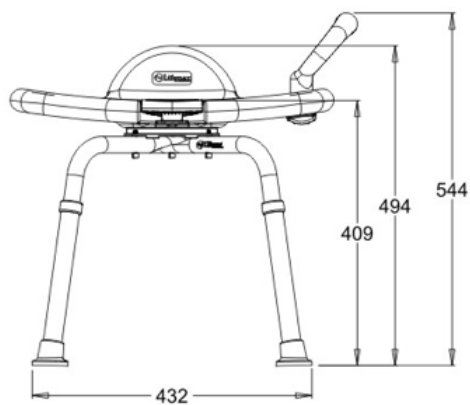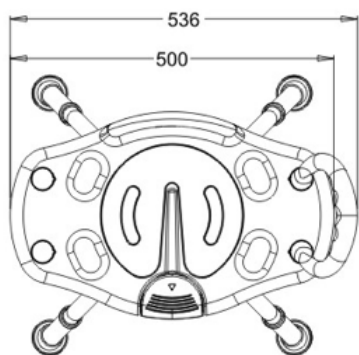Dimension RS702	 L: 595 mm	 W: 500 mm	 H: 682 mm
 Weight	4,28 kg		
 Max loading weight	136 kg		

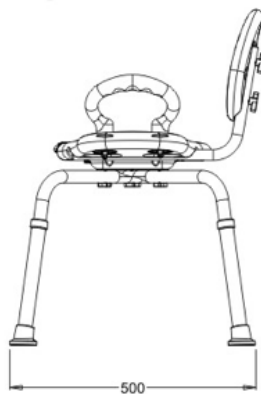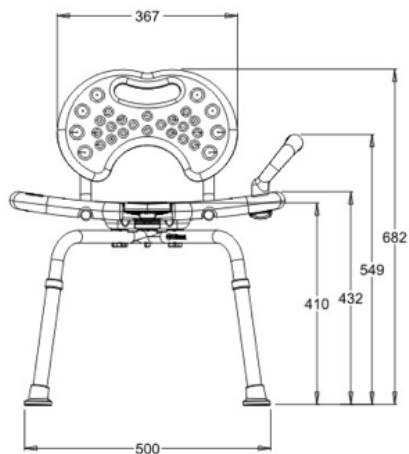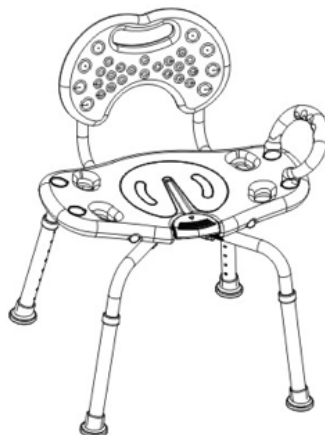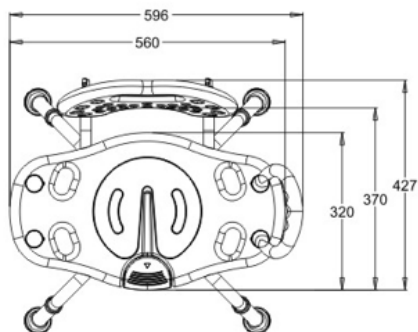Dimension RS705	 L: 800 mm	 W: 400 mm	 H: 30 mm
 Weight	4,43 kg		
 Max loading weight	136 kg		

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14.1 Repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

14.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

14.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com**MADE IN P.R.C.**

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com